

## 7.0 MAINTENANCE AND STORAGE

---

### 7.1 Maintenance of main body, handpiece and cartridge

#### 7.1.1 Main body and handpiece

- Clean the handpiece after each use.

##### How to clean:

- Use isopropyl alcohol and a soft cloth or isopropyl wipes to clean down the screen, main body and the handpiece.
- Dry with a clean, soft cloth.

##### NOTE

Ensure the device is turned off before cleaning or carrying out any maintenance and repairs to prevent serious damage to the user or to the device.

#### 7.1.2 Cartridge

- Always keep the cartridge at room temperature to prevent damage.
- The cartridge should be used within 6 months of the date of manufacture.
- Regularly check the level of water in the cartridges, top up as required.
- Cartridges that are to be stored for a long period of time should have the water removed. Deaerated water should be refrigerated.

#### 7.1.3 For device safety and longevity, keep away from:

✗ Water	✗ Direct sunlight	✗ Dust	✗ High humidity
✗ Chemicals or gas	✗ Salt	✗ Inadequate ventilation	