

## 6.2.4. Selecting Treatment mode – Manual or Auto mode

### Manual Operation

Press the button once, needle operate once

### Auto Operation

**Start** - Press and hold the button more than 2 seconds

**Stop** - Press the button once

## 6.3 Storage

- Store the unit in an area without high temperature or humidity.
- Ensure the device is turned off before storing and unplug the main power cord.

# 7.0 CLEANING AND STERILISATION OF THE DEVICE

## 7.1 Cleaning the Unit Body

- When cleaning the unit body, turn the power switch off and remove the power cord before cleaning.
- Wipe unit body with lint free dry cloth.
- Inspect the unit body for any foreign substances or matter. Use a dry cloth to gently wipe with 70% Isopropyl Alcohol to properly disinfect affected areas.
- Keep device in a safe place after cleaning.

## 7.2 Cleaning of the Handpiece

- Before cleaning the handpiece, make sure to turn off the power switch and remove the power cord from the outlet.
- The microneedle cartridges are sterile and single use. They must be disposed of as per local government regulations. The microneedle cartridge should be properly disposed of in a medical sharps container after use.
- Disinfect the handpiece and cable, before and after each treatment. Use a soft cloth and isopropyl alcohol solution or isopropyl wipes.
- Ensure the surface remains damp for at least 5 minutes. An additional application may be required if the device dries too quickly.
- Place handpiece in holder after cleaning.

## 7.3 Microneedle Cartridge Tip

- Only use the microneedle cartridges as recommended.
- **Microneedle cartridges use insulated needles.**
- Only open the new, sterile microneedle cartridge when the treatment is ready to commence.
- Shelf life: 3 years.



### i

*The micro-needle cartridge tip is disposable and supplied sterile. It is intended for single use only. Do not attempt to reuse or re-sterilise the tip. The micro-needle cartridge tip should be discarded after each use to avoid infection.*