

# Elite+ Cleaning & Disinfecting Manual

---

## Cleaning Exterior Components

Cynosure suggests that operators periodically clean and disinfect the exterior of the laser system. Always turn off the system before cleaning.

### Cleaning System Exterior

1. Using a soft cloth, clean the exterior of the laser with a mild soap and water.
2. When necessary, using a soft cloth, disinfect the exterior parts of the equipment with a hospital-grade disinfectant.
3. Be sure to clean and disinfect the handpiece storage cup on the rear of the laser.

### Cleaning Display

1. Prepare a mild soap and water solution.  
**CAUTION: Do not use any organic solvent, acid or alkali solution on the display or it may become damaged.**
2. Immerse a soft cloth into the solution, and then wring water from cloth.
3. Wipe the surface of the display with the dampened cloth.

### Cleaning the Cal Port

The window in the cal port must be kept clean to ensure proper operation.

**WARNING: Failure to keep the cal port window clean may result in incorrect fluence leading to patient injury.**

1. Between patients, examine the cal port for debris.
2. Clean the window using a lint-free cloth whenever debris is present.

## Cleaning and Disinfecting Handpiece Components

The handpiece holder, handpiece, handpiece tip and handpiece window should be cleaned and disinfected after each treatment session in the following manner:

1. Use a soft cloth for both cleaning and disinfecting.
  2. Clean with a mild soap and water.
  3. Be careful not to contaminate the optics with soap or disinfectant.
  4. Disinfect the handpiece by wiping the exterior surfaces, especially the tip, with a hospital-grade disinfectant.
  5. Disinfect the handpiece window using a 70% isopropyl alcohol wipe. Allow window to air dry thoroughly before using.
-