

SculpSure Cleaning & Disinfecting Manual

Care and Handling: System Exterior

Cynosure suggests that operators periodically clean the exterior of the laser system. Always turn off the system before cleaning. When cleaning, wear protective clothing and eyewear according to the cleaner manufacturer's instructions.

Cleaning System Exterior, Umbilical, and Dock (non-client contacting parts).

1. Using a soft cloth, clean the exterior surfaces of the System as well as the Umbilical and Dock with a mild soap and water.
2. When necessary, clean with 70% isopropyl alcohol.

Cleaning Display Screen.

1. Prepare a mild soap and water solution.



CAUTION: Do not use any organic solvent, acid or alkali solution on the display or it may become damaged.

2. Immerse a soft cloth into the solution, and then wring excess from cloth.
3. Wipe the surface of the display with the dampened cloth.

Cleaning Belt and Buckles.

1. Clean the belt and buckles after each use.
2. After use, remove the Belt from the Frame and rinse the parts with warm tap water to remove any visible debris. Do not remove buckles from belt.
3. Soak the belt and buckles for at least 15 minutes in 70% isopropyl alcohol (isopropanol).
4. Remove the belt and buckles from the cleaning solution and allow the parts to dry completely, for at least 15 minutes, before use.

Cleaning Pause Button.

1. Clean the Pause Button after each use.
2. After use, wipe the client-contacting surfaces of the Pause Button with a clean wipe dampened in 70% isopropyl alcohol (isopropanol), such as Steris Septihol Sterile Alcohol Solution, according to the manufacturer's instructions. Discard the soiled wipe.
3. Next, dampen a clean wipe with 70% isopropyl alcohol (isopropanol), such as Steris Septihol Sterile Alcohol Solution, and wipe the client-contacting surface thoroughly for two (2) minutes. Follow this cleaning with an additional eight (8) minutes of continuous contact of the wipe with the surfaces being cleaned—this may be accomplished with wrapping the wipe around the client-contacting surfaces. Discard the wipe after use.
4. With an unaided eye, visually examine the surface to ensure all adherent soil has been removed. Repeat the cleaning process if necessary.
5. Allow Pause Button surfaces to dry completely before use by either air-drying or with a lint-free tissue.

(Continued)

Cleaning Submental Mask


1. Thoroughly clean the mask after each use.
2. After use, disassemble mask and frame. See below for cleaning instructions for frame.
3. Rinse with warm tap water to remove any visible debris.
4. Thoroughly wipe the surfaces of the mask with a clean wipe dampened with 70% isopropyl alcohol (isopropanol) for at least 30 seconds.
5. Allow to air dry.

Disinfecting Submental Mask

1. After disassembly, thoroughly clean the mask after each use as directed above.
2. Thoroughly wipe the surfaces of the part with a clean wipe dampened with 70% isopropyl alcohol (isopropanol) for at least 1 minute for disinfection.
3. Allow to air dry.

Cleaning Frames

1. Thoroughly clean each frame after each use.
2. After use, disassemble mask and frame and rinse the frame with warm tap water to remove any visible debris.
3. Soak the parts in warm tap water for at least 5 minutes.
4. Using supplied brush, brush the frame/template while keeping it under water, with particular attention to all joints and spaces.
5. Remove the parts from the water soak and rinse again with warm tap water.
6. Wipe all surfaces with a clean wipe dampened with 70% isopropyl alcohol (isopropanol), and allow parts to air dry.

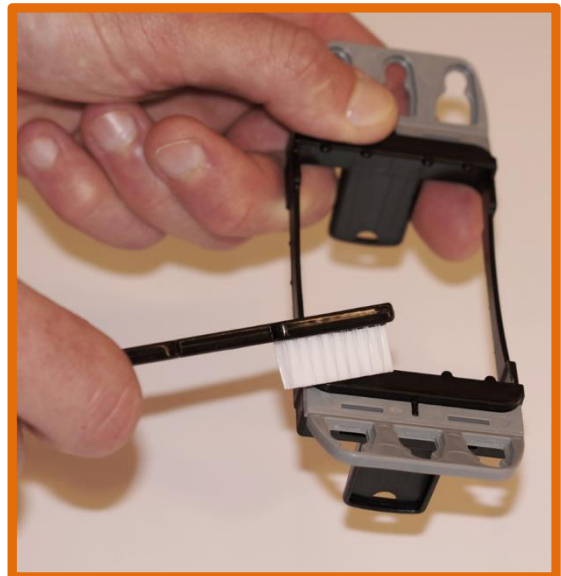

Disinfecting Frames

1. After disassembly, thoroughly clean the parts after each use as directed above.
2. Soak the cleaned parts for at least 15 minutes in 70% isopropyl alcohol (isopropanol).
3. Remove the parts from the solution and allow to air dry completely, for at least 15 minutes, before use.

(Continued)


Cleaning Applicator

1. Thoroughly clean the client-contacting surfaces of the applicator after each use.
2. Wet the supplied brush with warm tap water, shake off excess water, and gently brush the surface of the applicator, with particular attention to the crevices surrounding the grey contact sense blocks.
3. Dampen a clean wipe with warm tap water and thoroughly wipe the client-contacting surfaces of the applicator for at least 30 seconds. Note that wipe should be damp, not wet.
4. Dampen a clean wipe with 70% isopropyl alcohol (isopropanol), and thoroughly wipe the client-contacting surfaces of the applicator for at least 30 seconds.
5. Allow applicator surfaces to dry completely before use by either air-drying or with an absorbent lint-free tissue.



Disinfecting Applicator

1. After use, wipe the client-contacting surfaces of the Applicator with a clean wipe dampened in 70% isopropyl alcohol (isopropanol), such as Steris Septihol Sterile Alcohol Solution, according to the manufacturer's instructions. Discard the soiled wipe.
2. Next, dampen a clean wipe with 70% isopropyl alcohol (isopropanol), such as Steris Septihol Sterile Alcohol Solution, and wipe the client-contacting surface thoroughly for two (2) minutes. Follow this cleaning with an additional eight (8) minutes of continuous contact of the wipe with the surfaces being cleaned. Discard the wipe after use.
3. With an unaided eye, visually examine the surface to ensure all adherent soil has been removed. Repeat the cleaning process if necessary.
4. Allow Applicator surfaces to dry completely before use by either air-drying or with a lint-free tissue.

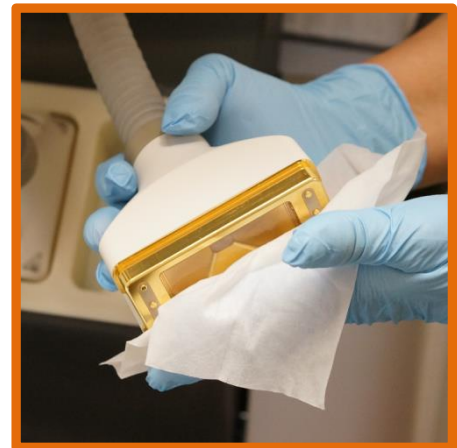

(Continued)

Cleaning Dock.

1. Clean the Dock after each use.
2. After use, wipe the inside and outside Dock surfaces with a clean wipe dampened in 70% isopropyl alcohol (isopropanol), such as Steris Septihol Sterile Alcohol Solution, according to the manufacturer's instructions. Discard the soiled wipe.
3. Next, dampen a clean wipe with 70% isopropyl alcohol (isopropanol), such as Steris Septihol Sterile Alcohol Solution, and wipe the inside Dock surface thoroughly for two (2) minutes. Discard the wipe after use.
4. With an unaided eye, visually examine the surface to ensure all adherent soil has been removed. Repeat the cleaning process if necessary.
5. Allow Dock surfaces to dry completely before inserting Applicator by either air-drying or with a lint-free tissue.